



Recommended use

1. Variable penetration

Dial the appropriate depth on the device between 1 (min.) and 5 (max.).

2. Simple and quick activation and reactivation

Pull back the grey slider to activate the mechanism.

3. Instant operation

Depress the grey button to operate the device.

4. Lancet removal

Simple and reliable lancet ejection.

Vitrex Compact Eject

Vitrex Compact Eject is a lancing device with a push button ejection. The device is clear adjustable with clear fixed end caps designed for Alternate Site Testing (AST).

VERSATILE

Suitable for use with Vitrex Soft Lancets and other most commonly used lancet brands.

VARIABLE PENETRATION

Nine depth settings from 1 to 5.

CONVENIENT

Simple and quick activation, reactivation and non-touch lancet removal.

Technical Specifications

Product	Description	Packaging	Article No.
Vitrex Compact Eject	9-step adjustable depth	Clear plastic sleeve only	369613

Shelf life 5 years

